

airCentric™ 2

How to add a headrest

Tools required:

- Power drill
- 3/8" drill bit
- Measuring tape

Note: All measurements should be taken from the edge of the plastic, not the foam.

Step 1

Measure the top of the airCentric back cap to find the center of the cap.

Step 2

Once center has been found, measure down 1.75".

Step 3

From the 1.75" location, measure 2.25" to the left and to the right of the center mark.

Step 4

Drill the 0.375" holes in the plastic cap.

Step 5

Align the new headrest bracket and install the two M6x25mm pan head fasteners provided, securing the headrest to the back of the chair.

